

ADE

Operating Manual
Electronic Lifter Scale M70611



Installation and use of scale

Your ADE scale is a quality product. In order to achieve best possible results please follow strictly the instructions of this Operating Manual:

Check carton contents

Electronic Lifter Scale
2 Safety hooks (1 snap link, 1 stable S-hook)
3 x AAA Batteries
Operating Manual

Insert batteries

- Open the battery compartment on the backside of the scale by sliding the lid aside.
- Insert 3 x AAA Batteries respecting the pole markings.
- Shut the battery compartment carefully and make sure it is completely closed.
- In case display shows 'LO' batteries are flat and need to be replaced.

Installation of scale to a suitable patient lifter

Insert the snap link into the upper the S-hook into the lower suspension of the scale.

- Remove the suspension yoke of the patient lifter and replace this by using the upper suspension eye of the scale to combine lifting arm and scale.
- Slide the suspension respectively the carrying equipment of the patient lifter carefully into the S-hook.

Display symbols and function keys

Underneath the display you are finding the following 3 keys:

- **ON/OFF** to start the scale and turn it off.
- **ZERO/TARE** to set the scale back to zero.
- **UNIT** key to change the weighing mode. You have the option to weigh either in kg or lb.

Weighing

To achieve a reliable result make sure the scale is always in vertical position. The S-hook must not yet be loaded by any weight.

Now press the **ON/OFF** key to start the scale. After a short function test the measuring becomes stable, the scale found its zero position and enters the '0.00' kg mode.

Loading the scale

Place the carrying equipment of the patient lifter carefully on the S-hook.

Place the patient in the carrying equipment (belt, weighing blanket or other) and make sure the patient is safely secured.

Tare function

Always use the Tare key if you are weighing the patient together with other objects (carrying equipment or other).

First weight the objects. After the weight has been displayed press the **Tare** key. The display will come back to zero '0.00'. The weight of the objects which have been weighed and memorized before will now be deducted from the all weighing procedures following until the scale turns off.

Place the patient into the carrying equipment like explained above.

The display is showing the weight of the patient.

Is the patient weight exceeding the maximum capacity (200 kg) the display will show ,E' which means the scale is overloaded.

HOLD-Function

By pressing the **HOLD** key the indication in the display will be hold for 5 seconds even if the weight value will change meanwhile.

Auto OFF

The scale is turning off automatically after 4 minutes however, also can be turned off manually at any time by pressing the **ON/OFF** key.

Cleaning & Maintenance

Safety instructions:

- Your ADE Lifter Scale is a precision instrument which needs to be handled with care.
- When using the scale please always follow thoroughly the Operating Manual.
- Make sure no water or cleaning fluid is penetrating the scale.

Cleaning:

For cleaning just use a damp cloth or common disinfection products.
Do not use any aggressive cleaning fluids.

Warranty – Liability:

ADE will repair or replace this product within a period of 2 years from date of delivery applying to defects occurred due to poor material or workmanship (presentation of invoice is necessary). All moveable parts – batteries, cables, mains units, rechargeable batteries etc. – are excluded. Warranty does not include natural wear, damages caused by incorrect or negligent handling, overuse or unsuitable or improper storage. The cost of transport will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be accepted subject to using the complete original packaging and the equipment secured the way it was packed and secured originally. All the packaging should therefore be retained.

Any warranty claim will be rejected if the equipment is opened by persons not authorized expressively by ADE to do so.

To make sure the scale is fully reliable at any time do check the scale in regular intervals by using tested weights.

In case of deficiencies:

Make sure the batteries have been inserted correctly.
Do not load the scale before the display shows ,0.00'.

Storage and Transport

Please keep all original parts and pieces for an eventual sending back of the scale in order to avoid any possible damage during transport which automatically leads to non warranty.

Declaration of CE Conformity

This product corresponds to the valid EC Directive 89/336/EEC for electromagnetic compatibility.



This product corresponds to the valid EC Directive 89/336/EEC for electromagnetic compatibility.

Technical Details

- Capacity: 200 kg
- Graduation: 100 g
- Abmessung: 80 x 150 x 45 mm
- Eigengewicht: ca. 0,5 kg
- Batteriebetrieb: 3 AAA
- LCD-Display
- Automatic zero setting, automatic shut off
- Very simple display
- Adoption to all leading patient lifter models

Supplier:

ADE GmbH & Co.
Hammer Steindamm 27 – 29
D – 22089 Hamburg/ Germany
☎ +49(40)432776-0
Fax +49(40)432776-10
E-mail: info@ade-hamburg.de

Dealer: